

Quick Guide for Installing Sidexis 2.5 on network without dedicated server.

DO NOT use these instructions if you are using a REAL SERVER such as Small Business, Server 2000, 2003 or 2008.

Preparation:

- In a multi-computer office, the Network must be up and running and the names of the computers final! Changing the names, IP schema or workgroup name later, will lead to malfunctions!
- Disable firewalls and virus scanners while installing.
- Create and add all users accessing the data in the network on the PC hosting SQL and the PDATA folder.
- Galileos installs: Create user: CBAAdmin with password: galileos838 with administrator rights on the station that holds PDATA and SQL.
- Create a folder named PDATA where the Images are to be saved. (on the largest partition/drive)
Give full control to everyone in "sharing" and "security" tabs.
Name of computer with PDATA: _____
- You must be logged in as the administrator on the PC to install the software.

Start on the Computer that will host SQL Express Server. NOT on a Windows SERVER operating system!

See the Sidexis Installation disk sleeve for approved Windows Operating Systems and hardware specifications.

NOTE: Less processing power may cause performance related issues during the accessing, loading and saving images.

CHECK FOR EXISTING SQL SERVER BEFORE INSTALLING SQL EXPRESS ONTO A COMPUTER!

- If no SQL Server is installed, proceed to step 1, below.
- * If MSSQL.1 folder is present under C:\Program Files\Microsoft SQL Server, create MSSQL.2 (or higher, as needed) folder structure (C:\Program Files\Microsoft SQL Server\MSSQL.2\MSSQL\DATA) Browse or enter the path during the install, at "SQL Server directories".

1. Insert the Sidexis XG CD into the workstation that will host SQL

The CD will either start automatically or you will need to run autorun.exe manually from the CD.

Select: Sidexis XG -> Sidexis XG Multistation. Choose SQL Server 2005 Express.

- Do not change defaults unless you had to create the MSSQL.2 folder as described above!
- Click: Next- Be patient. The installation may take several minutes.

Done

2. After SQL installation is finished, select: SIDEXIS XG, then select: "Central Installation".

- Select the country and accept the license agreement.
- Enter Full Name of Client, Practice Name and the serial number from tab in the disk.
All fields are MANDATORY! **DO NOT select "install viewer"!**
- Select the SQL Server in the drop down box: "YourComputerName\PDATA_SQLEXPRESS".
*If no database can be found, enter the name of the computer you are working on. (Where you installed SQL) Click next.
- For the image location field: Enter the UNC path "\\servername\pdata" for the PDATA folder you created in preparation, ("servername" is the name you wrote in preparation)
Be patient. The installation may take several minutes.
- At the end of the install, select to register "Later".

Done

3. After the install, open Sidexis and open an image. (Hit <ENTER> 4 times)

Done

4. In Utilities -> Configure System > Permanent Dentist: Edit "Dr. Demo" to the correct name.

Done

Sidexis Central install is finished.

Continue with the install of clients, add devices, add 3D software (Quick Guide available) and set up NGSQBACKUP.

- NGSQBackup.exe is found in PDATA\setup\tools\NSSQBACKUP folder. Instructions in "readme" or Quick Guide.

Sirona cannot be responsible for a proper backup!

Install Clients: DO NOT USE ANY CD ROM TO INSTALL CLIENTS!

- Start->Run and enter: \\servername\pdata\setup\autorun.exe
- Select Sidexis XG and accept the defaults.
- After completing the installation, start Sidexis and open an image. (Hit <ENTER>4 times)

Don't use with WINDOWS SERVER operating system

Don't use with WINDOWS SERVER operating system