Neu ab:
New as of:
Nouveau depuis:
Nuevo a partir de:
Nuovo a partire da:

06.2018

Sidexis 4 Viewer

Version 4.3

Installationsanleitung
Installation Instructions
Manuel de l'utilisateur
Instrucciones de instalación
Istruzioni di installazione

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1 The Sidexis 4 Viewer


The SIDEXIS 4 Viewer is used to examine X-ray images and findings from Sidexis 4 if the full version of Sidexis 4 is not available.

The images are examined in the same way as with Sidexis 4, but the functionality is severely limited in comparison with the full version. The following functions are not available in SIDEXIS 4 Viewer:

- Exporting media
- Connecting to any type of unit (X-ray units, film scanners, treatment centers)
- Deleting patients and media
- Connecting to a practice management system (via SLIDA interface)
- Connecting to GALILEOS Implant
- Connecting to a clinical PACS
(compatibility with Sidicom WLS, Sidicom QR, Sidicom PS)

2 No Sidexis 4 Viewer or Sidexis 4 installed


- ✓ Neither Sidexis 4 nor SIDEXIS 4 Viewer are installed on the computer.
- 1. Insert the DVD into the computer's drive.
 - ↳ The CD's contents are shown in the directory.
- 2. Click on the file "Start Sidexis 4 Viewer.exe" with the mouse.
 - ↳ NOTE: If no "Microsoft.NET Framework" is installed on your computer, a dialog box will open. In this case click on the "Accept and install" button and, once installation is complete, click on the "OK" button.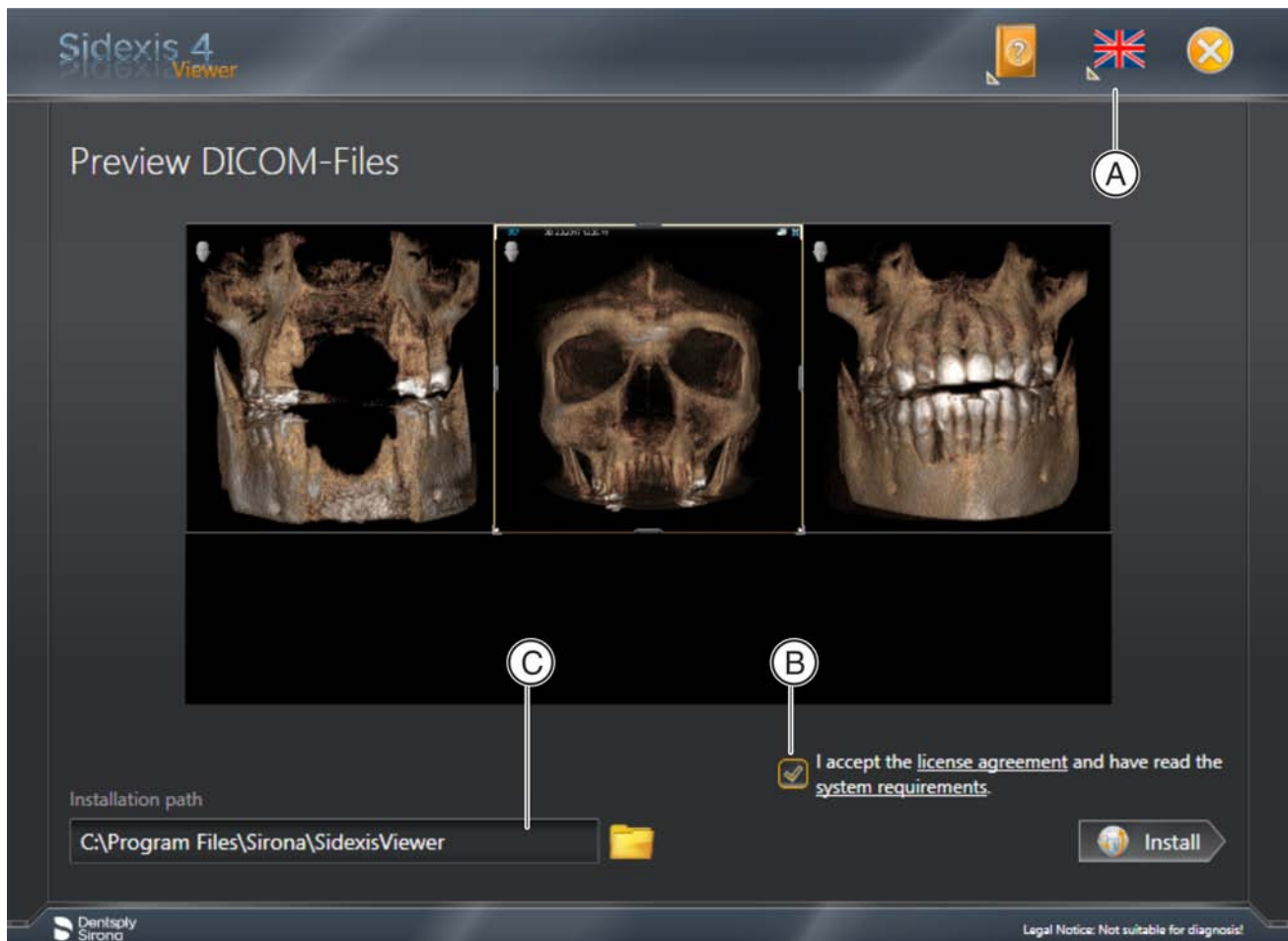

Installation menu for the SIDEXIS 4 Viewer

- ↳ The SIDEXIS 4 Viewer installation is prepared and the installation menu opens.
 - NOTE. In the installation menu, you are shown a preview of the data.
- 3. Should you so wish, you can change the language via the flag icon (A) in the SIDEXIS 4 Viewer header.
- 4. Click on the "System requirements" link and check them.
- 5. Click on the "License conditions" link. Read through the license conditions for SIDEXIS 4 Viewer and, if you accept the conditions, click on the checkbox (B).

6. Should you so wish, you can also change the installation path by entering it in the input window (C) or selecting the installation path by clicking on the folder icon.
7. Start the installation by clicking on the "Install" button.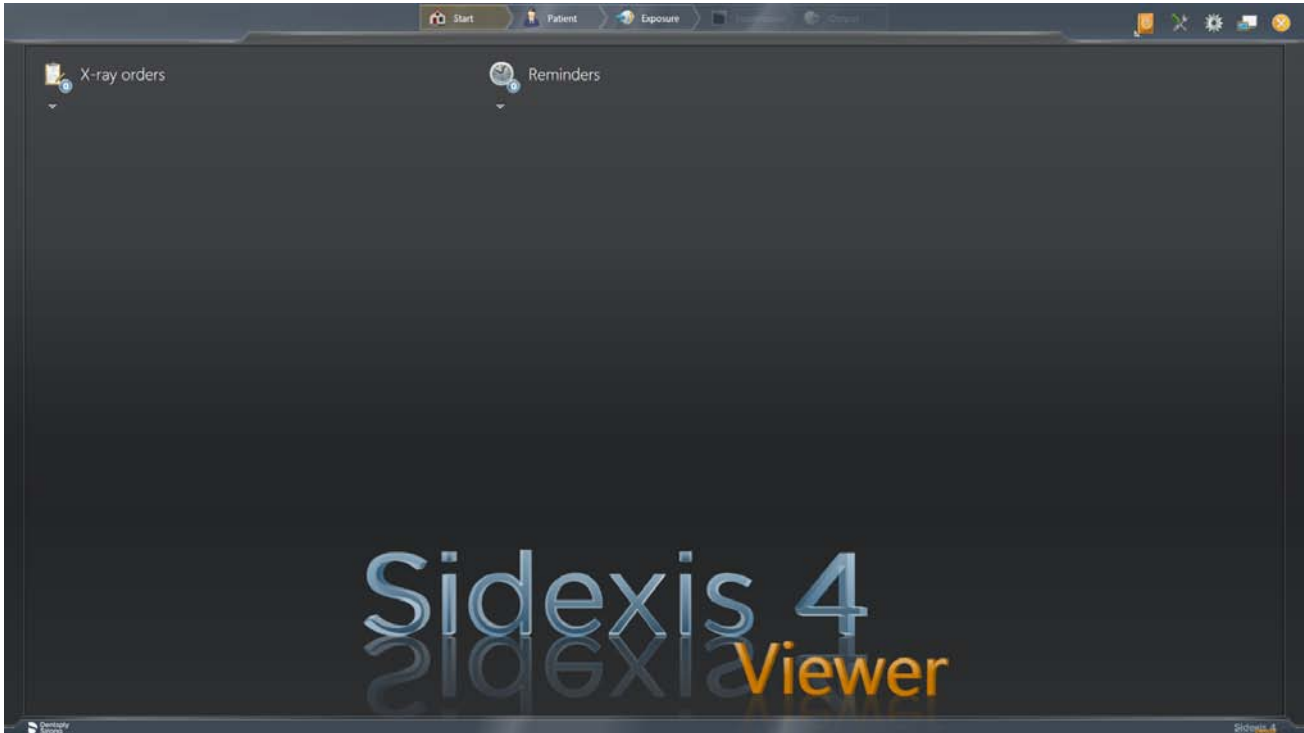


SIDEXIS 4 Viewer

- ↪ SIDEXIS 4 Viewer is installed and opened.
- ↪ After SIDEXIS 4 Viewer has opened, the import of the data records provided starts automatically.
IMPORTANT: This automatic import can take several minutes. Following the import, the data is automatically opened in the "Light box" of SIDEXIS 4 Viewer.


3 Sidexis 4 Viewer or Sidexis 4 available

- ✓ Sidexis 4 or SIDEXIS 4 Viewer is already installed on the computer.
- 1. Insert the DVD into the computer's drive.
 - ↳ The CD's contents are shown in the directory.
- 2. Click on the file "Start Sidexis 4 Viewer.exe" with the mouse.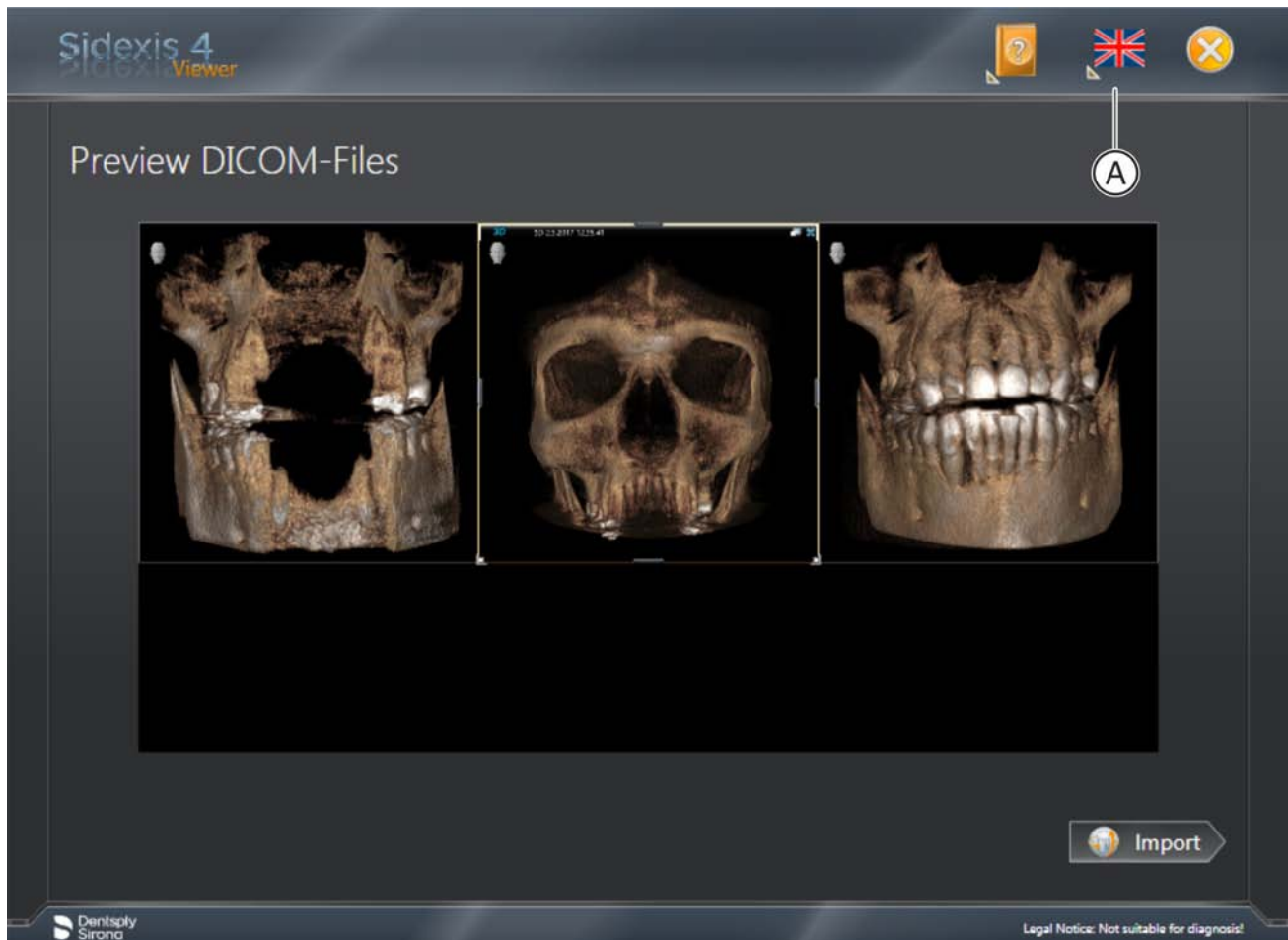

Import menu for the SIDEXIS 4 Viewer

- 3. Should you so wish, you can change the language via the flag symbol (A) in the SIDEXIS 4 Viewer header.
- 4. Start the import of data by clicking on the "Import" button.

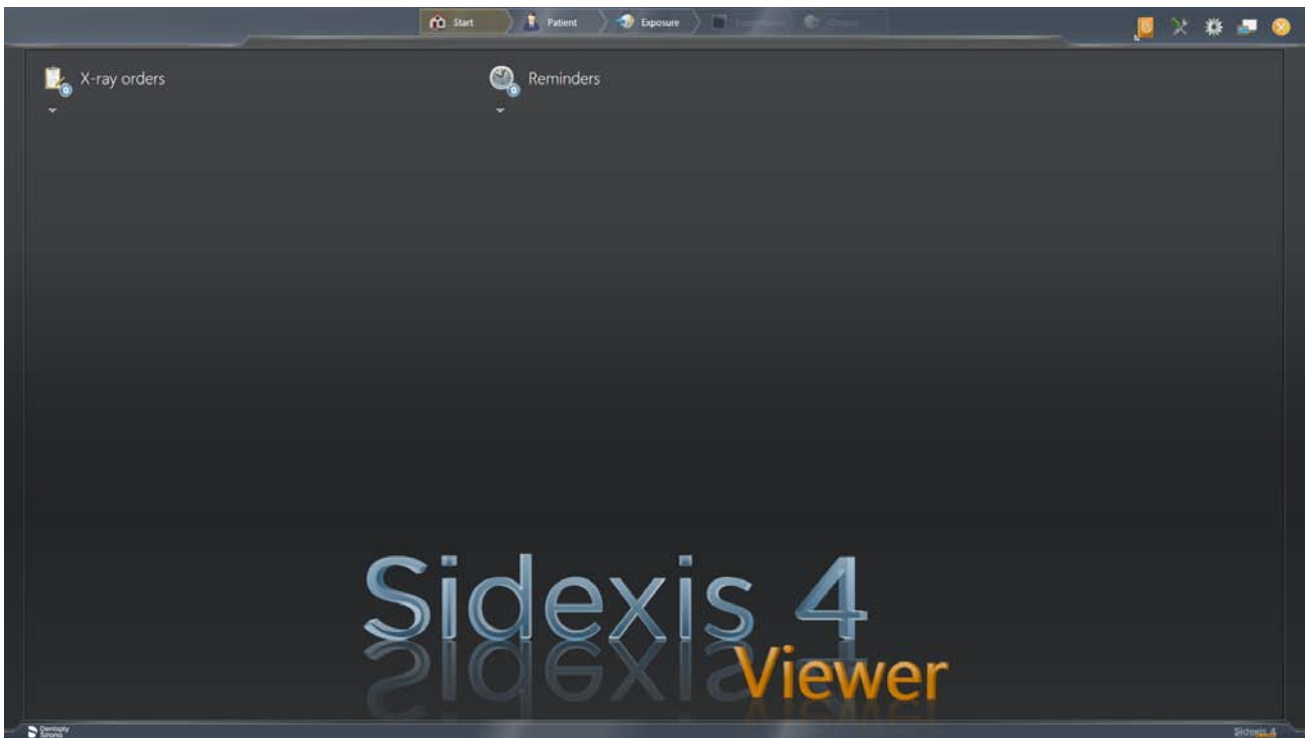SIDEXIS 4 Viewer

- ↳ SIDEXIS 4 Viewer opens.
- ↳ After SIDEXIS 4 Viewer has opened, the import of the data records provided starts automatically.
IMPORTANT: This automatic import can take several minutes. Following the import, the data is automatically opened in the "Light box" of SIDEXIS 4 Viewer.

NOTE: If Sidexis 4 is installed on the computer, the data is opened directly in the "Light box" of Sidexis 4.

We reserve the right to make any alterations which may be required due to technical improvements.

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© Sirona Dental Systems GmbH
Sirona Dental Systems GmbH