

# CDRDicom Exclusions for IT

Stop anti-virus & windows defender and have administrator rights to install CDRDicom.

Please add these exclusions to your security for CDRDicom to work in a secure environment

## Anti-Virus, Defender & Firewall Exceptions:

### File paths for exclusion

C:\Program Files (x86)\Schick Technologies

C:\Program Files (x86)\Sirona Dental

C:\ProgramData\Schick Technologies

C:\Program Files\Microsoft SQL Server

### Ports for exclusion

TCP 1433 default port for SQL Server

TCP 1434 default SQL port for the Dedicated Admin Connection.

UDP 1434 is used for SQL Server named instances.

### XML Parser on all machines

<https://www.microsoft.com/en-us/download/details.aspx?id=15697>

Choose msxml.msi for download