

This configuration of CDR objects does not permit query or retrieval.


This is due to a corrupt XML config file.

- 1) Navigate to <C:\ProgramData\Schick Technologies\CDR>
- 2) Rename CDRSettings.xml to: CDRSettings-old
- 3) Re-Launch CDRDicom; a new XML file will be re created.

You will need to reconnect to the database

Toolbar > system > database Options> Configure database Connection Tab.


From the pull down, Select the server you want to connect to.

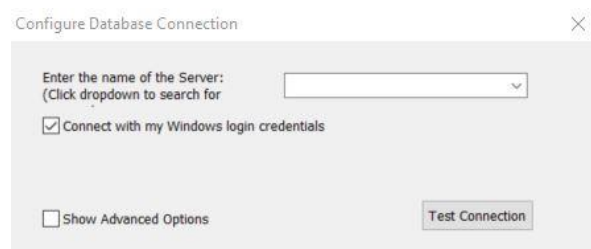

“In CDRDicom, you will need to add these modules”

“ Toolbar ”, select system> CDR Options > acquisition Modules tab > Click Add.

Repeat the add process for each module listed int this Image.

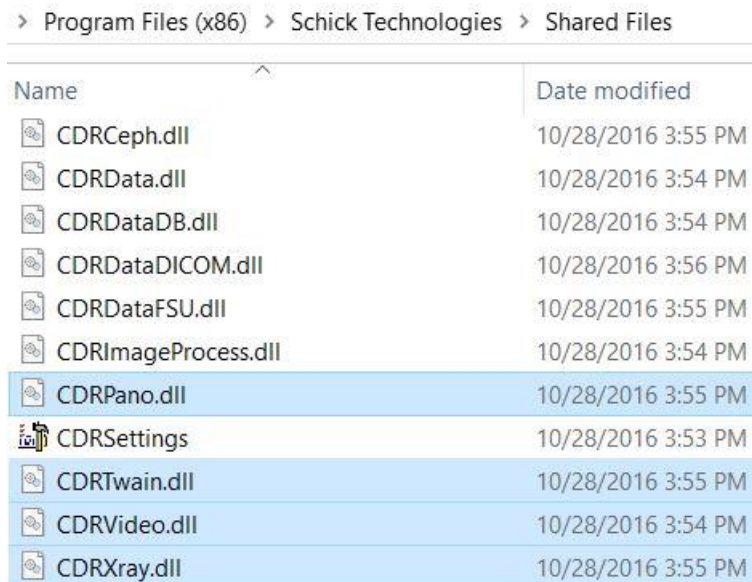

Once complete exit and restart the CDR Dicom application.

On Occasion you might need to reload the Drivers for the sensors