

# CDR Firmware Update

CDR Intraoral Imaging System from Schick Technologies, Inc.

## Instructions for CDR Customers

### INSTALLATION PROCEDURE

1. Exit CDR (or EagleSoft or Patterson Imaging) if it is running and verify the following:
  - ❑ CDR2000 Remote Module is NOT connected to laptop or desktop PC. If connected, disconnect the Remote Module by unplugging the USB cable.
2. Install Firmware 36 Software:
  - ❑ Insert the CDR Firmware 36 CD (supplied) into your CD or DVD drive.
  - ❑ Click Install Firmware 36 Software when the start page is displayed. If the page does not display, browse to setup.exe on CD and double-click the file to launch it.
  - ❑ Click Next at the Welcome Screen.
  - ❑ Click Install at the Ready to Install Program Screen.
  - ❑ Click Finish at the end of the installation.
  - ❑ If prompted, click Yes to restart the laptop or desktop PC and wait until the system restarts before continuing.
3. Upgrade Remote Module to Firmware 36:
  - ❑ Re-connect the Remote Module and any Sensor to your laptop or desktop PC.
  - ❑ Browse to your Desktop and double-click on the "Upgrade Firmware" shortcut, which starts the utility and upgrades the Remote Module to firmware 36 (about 10-15 seconds). Do not disconnect the Remote Module or the Sensor during the upgrade.
  - ❑ When complete, the utility closes automatically.
4. Start your imaging program and take several sample exposures:
  - ❑ Note: If CDR DICOM is your imaging program, refer to the CDR DICOM User Guide for instructions on taking X-rays. For other programs, refer to the procedures supplied by the manufacturer. In either case, please continue with the following steps.
  - ❑ Create a sample X-ray exam.
  - ❑ Place a radio-opaque object like a coin or clip on the active (flat) side of the Sensor.
  - ❑ Take several sample exposures and verify image quality.
5. Please repeat the preceding steps for every CDR2000 Remote Module in your office.

Schick Technologies Software Engineering / August 28, 2008

