

# *Installing USBCam2 64-Bit Beta Driver on Windows 7*

**(c) 2011 SCHICK TECHNOLOGIES, INC.**

**ALL RIGHTS RESERVED**

---

**PURPOSE**

The following information will assist you with installing the 64-bit beta driver for USBCam2 intraoral cameras on Windows 7.

**REQUIREMENTS**

These instructions assume that you have the following:

- Windows 7 Home, Premium, Business, or Ultimate (all 64-bit)
- CDR DICOM 4.5 CD or internet access to our website

**INSTALLATION  
PROCEDURE**

Perform the following to install the USBCam2 driver.


*Downloading the Driver*

1. Close CDR DICOM 4.5 software if open.
2. [\*Download the USBCam2 64-bit beta driver from our website.\*](#)
3. Click Run to start the setup program. If prompted, click Run again to confirm.

---


## Installation Procedure

4. Click Install (*Figure 1*).

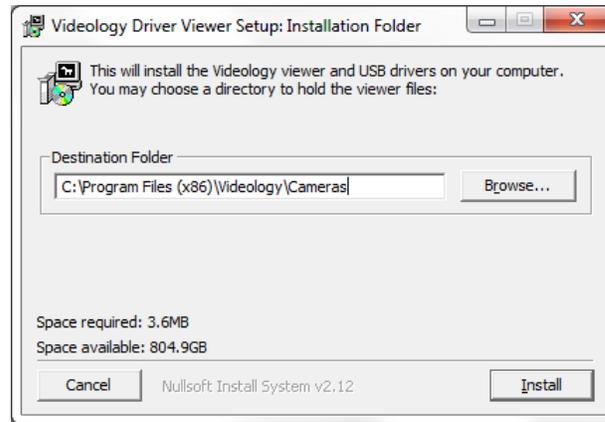

*Figure 1. Driver Download Setup*

5. After the files have been downloaded, click Close (*Figure 2*).

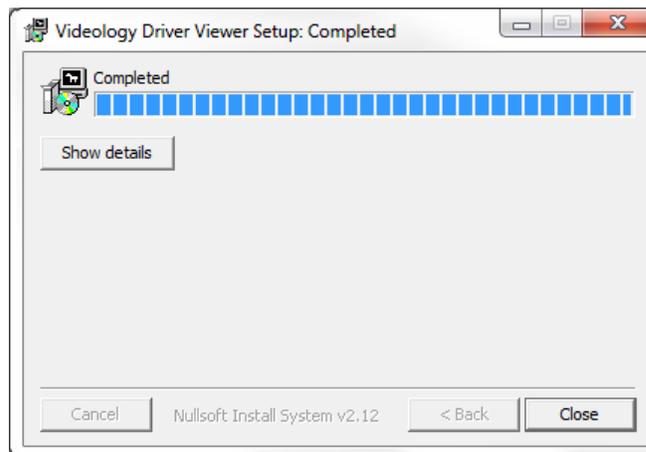

*Figure 2. Driver Download Complete*

6. Connect the USB cable to the USBCam2.


### *Updating the Driver*

1. Open the Windows Control Panel and navigate to System and Security > System > Device Manager.

---


## Installation Procedure

2. Browse for the USB 2820 Device in the Device Manager (*Figure 3*).

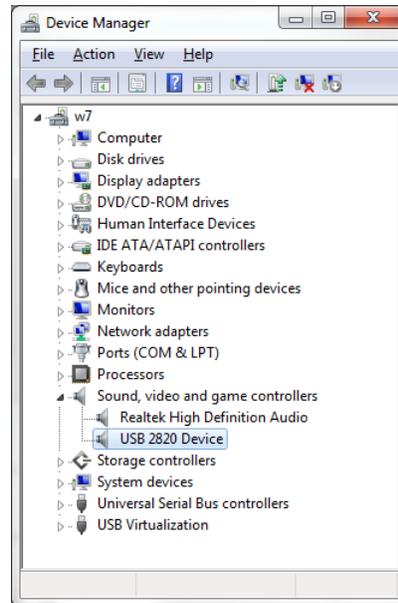

*Figure 3. Device Manager*

3. Right-click on the USB 2820 Device and select "Update Device Driver Software" (*Figure 4*).

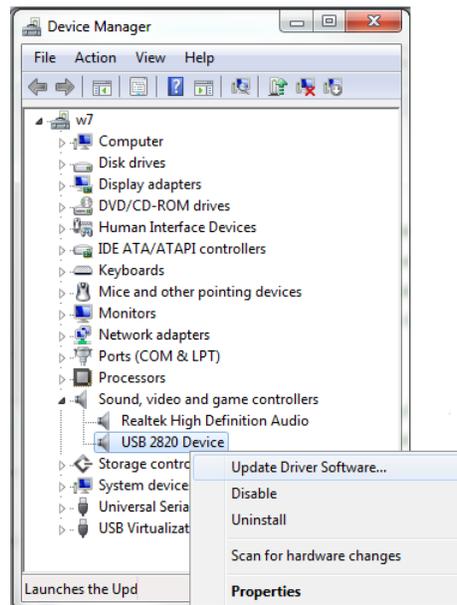

*Figure 4. Update USB 2820 Device Driver*

---

## Installation Procedure

4. Click “Browse my computer for driver software” (Figure 5).

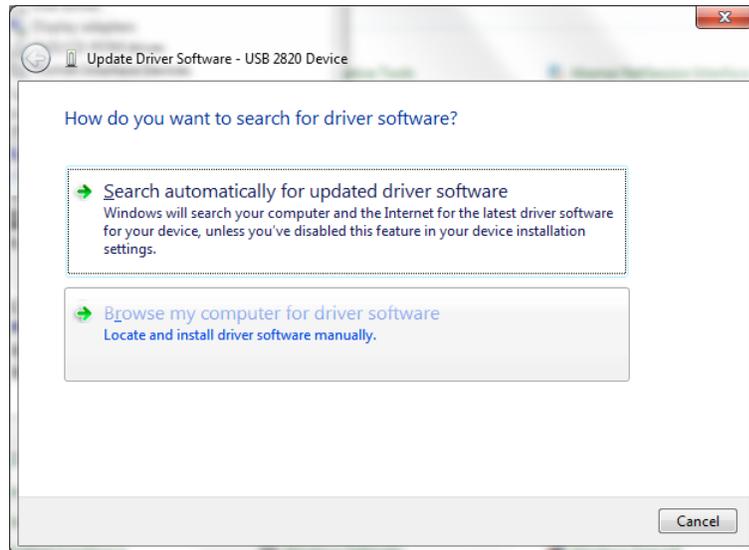


Figure 5. Browse for Driver Software

5. Click “Let me pick from a list of device drivers on my computer” (Figure 6).

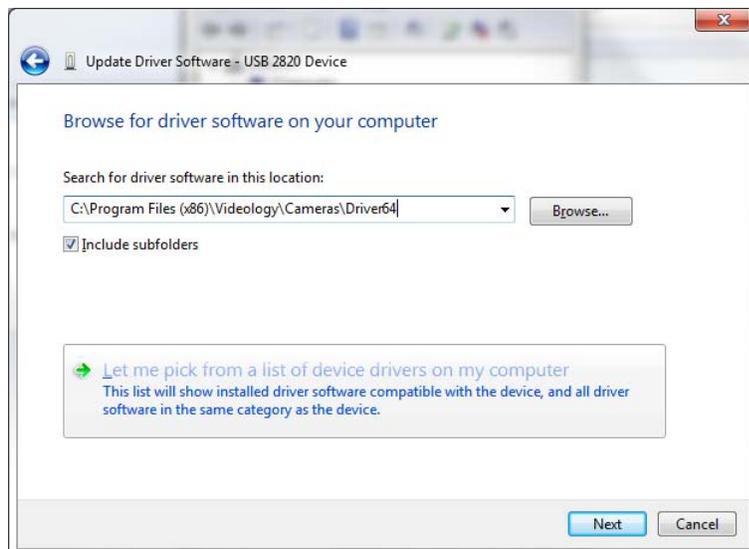


Figure 6. Show Compatible Driver Software

---

## Installation Procedure

6. Click “Have Disk” (Figure 7).

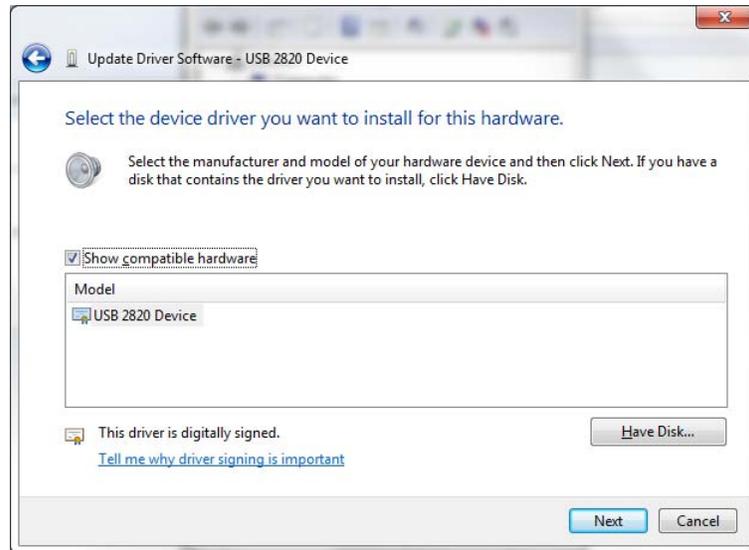


Figure 7. Select Driver Software

7. Click the Browse button and navigate to the 64-bit Videology driver folder:  
C:\Program Files\x86\Videology\Cameras\Driver64 (Figure 8).

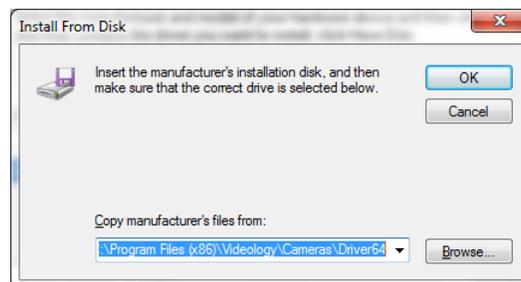



Figure 8. Copy Driver Software

8. Click OK.

---


## Installation Procedure

9. The Update Driver Software window displays the model as “Videology USB Camera” (*Figure 9*), and not USB 2820 as shown previously.

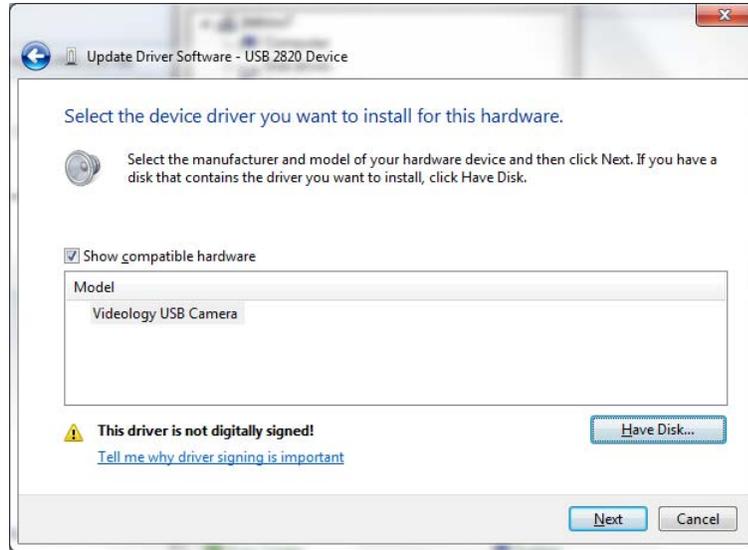

*Figure 9. Device Name Updated*

10. Click Next.
11. Click “Install this driver anyway”, if prompted (*Figure 10*).

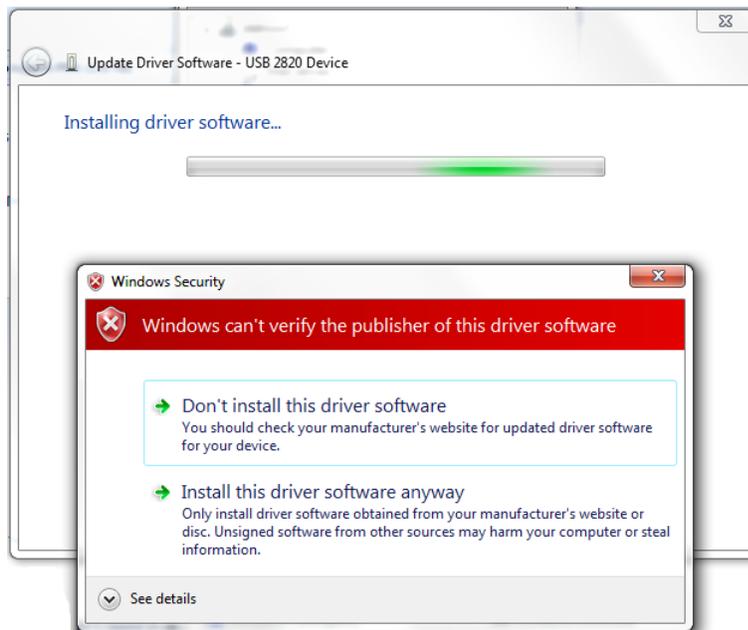


*Figure 10. Windows Security Message*

---

## Installation Procedure

12. The message, “Windows has successfully installed your software driver”, is displayed (*Figure 11*).

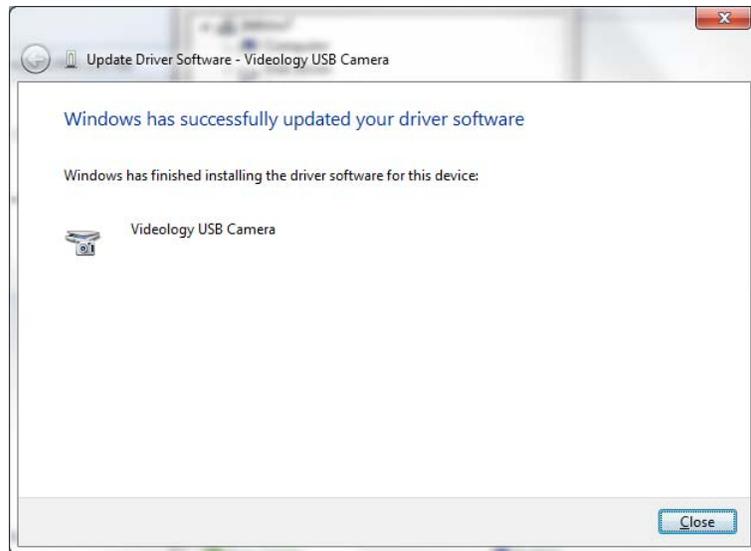

*Figure 11. Driver Software Update Successful*

13. If you have questions regarding the installation of the USBCam2 64-bit beta Driver on Windows 7, please contact your appropriate support center for Schick Technologies' products or the authorized international dealer in your country or region.