

Sending X-rays to Non-CDR Dicom Customers

(c) 2010 SCHICK TECHNOLOGIES, INC.

ALL RIGHTS RESERVED


PURPOSE

The following information will assist you with using the Web Viewer feature in CDR DICOM software to send X-rays to non-CDR Dicom customers.

PROCEDURE

1. Create an empty folder on your Windows Desktop, preferably with with the name of the patient. This folder will be used to store the images you intend to send.
2. Open CDR Dicom, if it's not running already, and open the patient's exam.


FIGURE 1. CDR Dicom patient exam

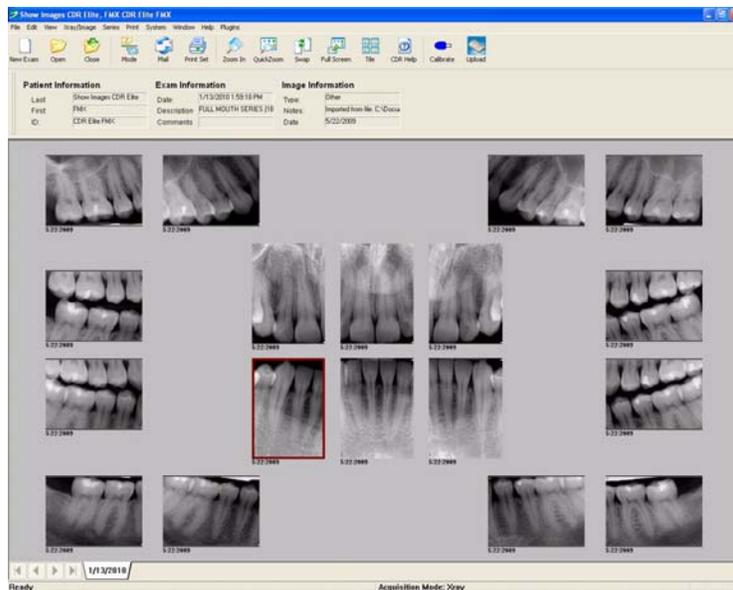

Procedure


- Click on File > Send To > Web Viewer.

FIGURE 2. Adding exam images to Web Viewer

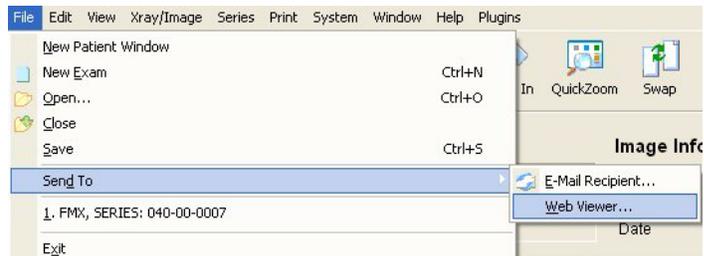

- When prompted, save the images in the desktop folder you created previously.

FIGURE 3. Saving exam images in Web Viewer

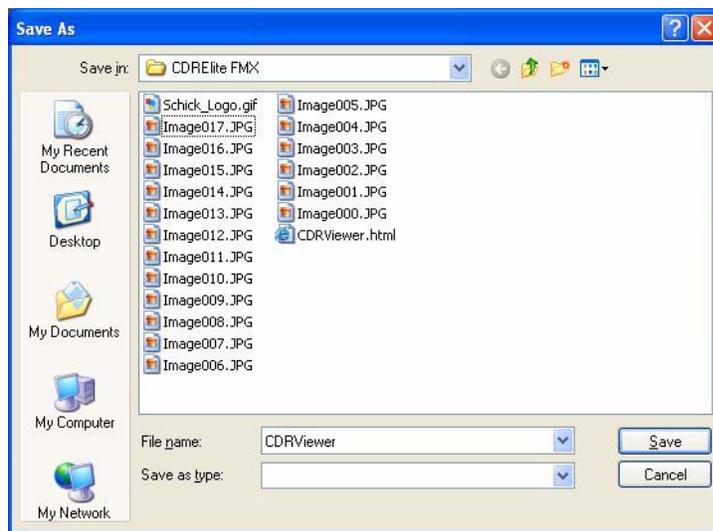

- When you've saved the images, the desktop folder will contain a "mini-website", consisting of a pre-formatted HTML page and all the images saved in JPG format. (For browser compatibility, the JPG format cannot be changed.) The folder can be zipped and attached to a mail message or copied to CD media.
- On receipt, the customer should double-click on the HTML page to display the exam and its images (see next figure for an example).

Please note that the recipient will need Internet Explorer 5 or higher to view the images inside the Web Viewer's exam page.

FIGURE 4. Sample Web Viewer exam

